



HELP STOP THE SPREAD OF COVID-19

OVER-THE-COUNTER RAPID SELF-TEST TO DETECT ACTIVE COVID-19 INFECTION

Panbio™ COVID-19 Antigen Self-Test detects infection in asymptomatic and symptomatic people.

NO AGE RESTRICTIONS

Approved for use on children under 14 years when supported by an adult.



TRUSTED BRAND IN MORE THAN 140 COUNTRIES



EASY

nasal self swab
procedure



FAST

results in
15 minutes



RELIABLE

test performance

EXCELLENT NASAL SWAB PERFORMANCE*


	SENSITIVITY	SPECIFICITY
PANBIO™ COVID-19 Ag RAPID TEST DEVICE NASAL SWAB PERFORMANCE COMPARED TO NASAL PCR WHEN PERFORMED BY A TRAINED HEALTHCARE PROFESSIONAL	98.1%	99.8%

HIGH-PERFORMANCE SELF-TEST DATA**

	POSITIVE AGREEMENT	NEGATIVE AGREEMENT
IN CLINICAL EVALUATIONS WITH SELF-TEST USERS, THE PANBIO™ COVID-19 ANTIGEN SELF-TEST CORRECTLY IDENTIFIED POSITIVE AND NEGATIVE SARS-CoV-2 SAMPLES**	95.2%	100%

EASY-TO-USE, SIMPLE PROCEDURE


1 PREPARE MATERIALS

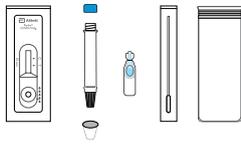

2 FILL TUBE




3 SWAB NOSE

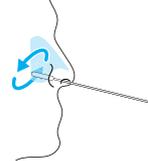


4 SWIRL SAMPLE



5 BREAK SWAB

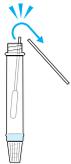


6 ADD 5 DROPS

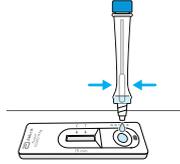

7 WAIT 15 MINUTES



8 READ RESULT

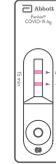

INDIVIDUALLY PACKED WITH INSTRUCTIONS FOR SELF-TESTING BY CONSUMERS

CATALOG NUMBER	41FK51 (1 TEST)	
	41FK91 (4 TESTS)	
	41FK71 (10 TESTS)	
	41FK81 (20 TESTS)	
KIT CONTENTS	<ul style="list-style-type: none"> • Test Device • Instructions for Use • Tube • Blue Cap 	<ul style="list-style-type: none"> • Buffer Bottle • Swab • Bag • Tube Rack



CONTACT YOUR LOCAL REPRESENTATIVE TODAY.

WWW.GLOBALPOINTOF CARE.ABBOTT

Product not available in all countries. Available to consumers in select markets. Not approved for sale in the USA.

*In medical diagnosis, sensitivity is the ability of a test to correctly identify those with the disease. Specificity is the ability of the test to correctly identify those without the disease. See product Instructions for Use for additional performance information.

**Positive and Negative agreement percentages demonstrate how well one test agrees with a second comparative test. The higher the percentage, the stronger the agreement between the two tests. Panbio™ COVID-19 Antigen Self-Test samples were confirmed as positive and negative by comparing to the Panbio™ COVID-19 Ag Rapid Test Device (Nasopharyngeal).

© 2021 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. Any photos displayed are for illustrative purposes only. COL-03422-01 05/21

